

# Microneedling Pen Step-By-Step

## Protocol

1. Before beginning the treatment, make sure you have your [SAIAN Pure Hyaluronic Acid Spray](#) and [SAIAN Cryo-Globes](#) in the refrigerator. The finishing cooling touch is very important at the end of this treatment.
2. Cleanse the face with [SAIAN Replenishing Cleanser](#). This milky cleanser's gentle and natural formula removes surface impurities and all traces of make-up, leaving skin moisturized and supple. Aloe vera and chamomile are hydrating, softening, calming, as well as anti-microbial and anti-inflammatory. This product is perfect for sensitive skin.
3. Apply numbing cream to treatment area and leave on for 30 minutes. Remove numbing cream thoroughly.
4. Apply antiseptic wipe to treatment area.
5. Apply 1/2 [SAIAN Hyaluronic Acid Ampoule](#) to the entire treatment area. The rest of the ampoule will be used later in the treatment.
6. Begin treatment by dividing the face into five parts: forehead, left and right cheeks, nose, upper lip and chin.
7. Apply a few drops of [SAIAN Hyaluronic Acid Ampoule](#) to the treatment area.
8. Perform microneedling according to your device's manufacturer's instructions. Work on all 5 sections in the same manner until the entire face has been treated. After treating each section, wipe any bleeding.
9. After the entire face has been treated, take the [SAIAN Pure Hyaluronic Acid Spray](#) out of the refrigerator, and spray the face with [SAIAN Pure Hyaluronic Acid Spray](#). This product serves as an activator for the collagen mask, as well as a light, oil-free hydrator.
10. Cover the face with [SAIAN Pure Pure Collagen Mask](#). Fold the mask in half over the client's face, mark and cut the mask around the nose area. Apply the dry collagen mask immediately over the face quickly and spray with more [SAIAN Pure Hyaluronic Acid Spray](#), molding gently to the face and neck with sponges. Leave mask on the face for no less than 20 minutes. Remove the mask.



11. Apply [SAIAN Vital K Creme](#) to the entire face. This creme provides a powerful dose of Vitamin K to repair the damaged area, supplying the patient's skin with the means to halt the flow of bruising and pause discoloration. To maximize and quicken the recovery process, we use Arnica. This natural herb, rich with anti-inflammatory and anti-bacterial properties is a highly efficient vasodilator of blood capillaries. Arnica drastically increases the fade back time, by escalating the recession of blood away from the damaged capillaries and back into the body. This helps return the skin to a flush and even color in a fraction of the time it would usually take after it has been traumatized by physical impact. Applying [SAIAN Vital K Creme](#) to the area significantly reduces the swelling and dark patch, and brings the color back to normal, as well as moves blood through the damaged capillaries for reduced discomfort and pain.
12. Spray with [SAIAN Sunscreen Mist](#). This is an oil-free light sunscreen spray. Chemical sunblocks are never recommended after invasive treatments, and this light Titanium dioxide based sunscreen is the perfect alternative.
13. Roll over the entire area with [SAIAN Cryo Globes](#).

