

A close-up photograph of a woman lying on a treatment table, her eyes closed in a relaxed state. A practitioner, wearing a white lab coat and white gloves, is using a handheld ultrasonic skin scrubber on the woman's cheek. The device is white with a black applicator tip. The background is softly blurred, showing a clinical or spa setting.



SAIAN[®]
NATURAL CLINICAL

Professional Strength
Ultrasonic Skin Scrubber


INSTRUCTIONS

Saian professional strength ultrasonic skin scrubber combines two facial treatments in one compact, hand-held, wireless machine. Used on moist skin, the deep cleaning mode exfoliates and thoroughly cleanses the face with ultrasonic vibrations that remove dead skin cells, dirt, and sebum. The skin immediately feels smooth, even, and soft.

The second mode uses sonophoresis for an enhanced application of Saian moisturizers, serums, and other anti-aging products.

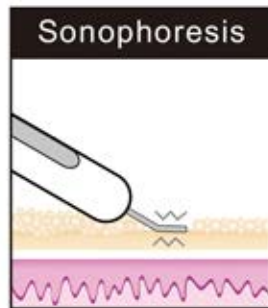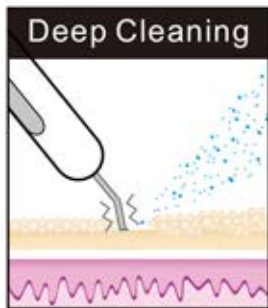

PARTS

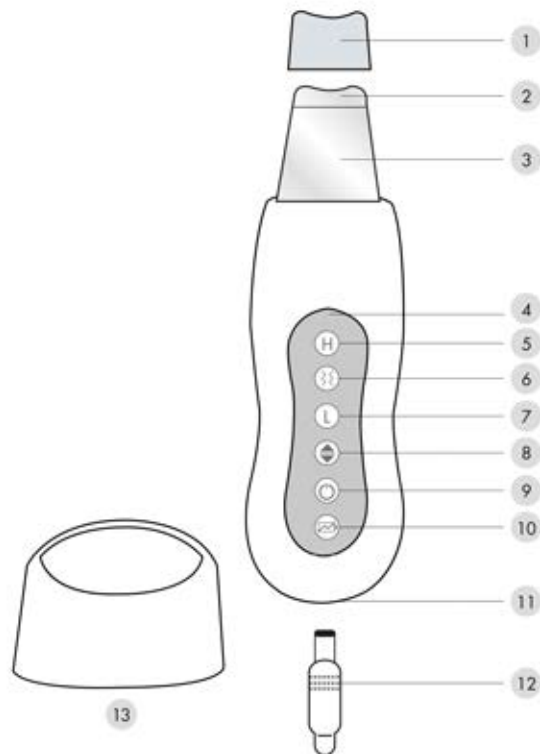

1. Protective sleeve
2. Blade tip
3. Ultrasonic blade
4. Control panel
5. High intensity mode indicator
6. Pulsing wave mode indicator
7. Low intensity mode indicator
8. Program selection button
9. On/Off button
10. Battery indicator light
11. Charging socket
12. Charging adaptor
13. Base

CHARGING YOUR DEVICE



Before using this device for the first time, charge it until the red charging light goes out. This may take 3-4 hours. When the battery is low, the machine will beep 6 times, and then automatically turn off.

1. Connect the charging adapter to the charging socket and plug into the wall.
2. When the unit is charging, the battery indicator will light up.
3. When the unit is fully charged, the battery indicator light will go out, and the machine is ready to be used.
4. After the charging has been completed, disconnect the charging adapter from the unit and unplug from the wall.

DEEP CLEANSING TREATMENT



Turn on the unit by pressing the On/Off button.



Use the program selection button to change the intensity of the device.

LOW intensity continuous wave mode

The low intensity continuous wave mode is automatically selected when you turn on the device. We recommend starting treatments on the low intensity program.



HIGH intensity continuous wave mode

Press the program selection button twice to select the high intensity continuous wave program. When this program is selected, the H light will illuminate blue.



DEEP CLEANING TREATMENT



1. Apply warm water to the face.



2. Turn on the unit by pressing the On/Off button.



3. Select Low/High/Continuous wave program.

5. Saturate a cotton round with water and hold the unit so that the control panel is away from the face. Cleanse the face according to the diagram on the following page. Treatment time is typically 3-5 minutes 3/week.



6. After use, turn the unit off by pressing the On/Off button again. If not turned off manually, the unit will shut off automatically after 10 minutes.

DEEP CLEANING TREATMENT



We recommend using the device according to this diagram, as it encourages the body's natural lymphatic drainage.

1. Sweep the machine from the neck upwards onto the face in horizontal strokes. Repeat on both sides of the face.
2. Methodically work over all areas of the face as shown in the illustration.
3. Change the direction of the handset and glide in vertical strokes starting from the neck area. Work from the jaw to the hairline.
4. Cleanse around the curve of the nostril using the curved inner edge of the blade.

TIPS FOR EFFECTIVE CLEANSING



Use the scooped out middle section of the blade to cleanse the chin area under the lip with vertical strokes.



Select the low setting to work around the eyebrow area. Avoid the delicate under eye area when using the high setting on the machine.



SONOPHORESIS



1. Turn on the unit by pressing the On/Off button.



2. Press the program selection button once to use the device in LOW intensity pulsing wave mode, or 3 times to use the device in HIGH intensity pulsing wave mode.



LOW intensity pulsing wave mode

When this program is selected, you will hear intermittent sonic vibrations, the L will illuminate blue, and the pulsing wave light will flash.




HIGH intensity pulsing wave mode

When this program is selected, you will hear intermittent sonic vibrations, the H will illuminate blue, and the pulsing wave light will flash.




SONOPHORESIS TREATMENT

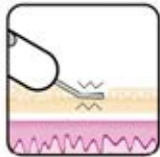

1. Apply your favorite Saian serum or moisturizer. Hold the device so that the blade is directly against the skin.
2. Turn the unit on by pressing the On/Off button. Select low / high intensity pulsing wave program. We recommend starting on the low intensity program.
3. Using gentle pressure, glide the blade over the surface of the skin using backwards movements. For best results, work slowly moving the blade at all times.
4. Turn the device off after 3-5 minutes, if left on, the device will automatically shut off after 10 minutes.

TIPS FOR BEST APPLICATION

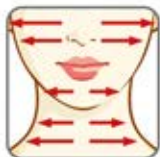

Beginning on the neck area, with gentle pressure, glide the blade over the surface of the skin, using a backwards movement, working from the center of the face outwards.



Continue upwards from the jawline to the hairline. Sweep over the face until all product is absorbed.



IMPORTANT INFORMATION

DO NOT immerse this product in water.

This product is NOT water resistant, do not place it anywhere where it can fall into water.

After each use, clean the stainless steel spatula with water or a cotton pad saturated with disinfectant.

DO NOT place this product near heat or an open flame.

DO NOT use this product if it's not operating correctly, or has been damaged.

There are no serviceable parts inside the unit.

Misuse of this product can be harmful.

CUSTOMER CARE

If you have any questions regarding this device, or if you'd like to order skincare products, please call us at 1-800-291-1130 and visit our site at www.saian.net

PRODUCT SPECIFICATIONS

Item number	859538008039
Power supply	AC100-240V
Output	DC : 5V, 0.8A
Ultrasound Frequency	25 KHz
Dimensions	17.1x5.2x2.1cm
Weight	97g

